


envo[®] mask


Quarter Mask Size Guide

Sizing Instructions


Print out “L-Shaped Template” to the actual size provided (100% scale) and cut the measuring tool along the dashed lines.

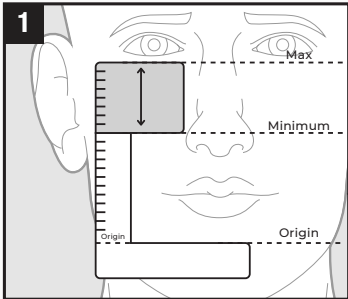


Using the provided L-shaped measuring tool, place the Origin line at the crease of your chin just below your lower lip. The tool should make an L-shape when applying it to your face.

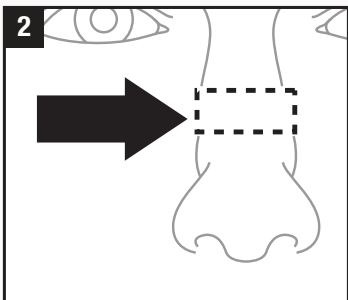


Find the area of your nose where bone meets cartilage.

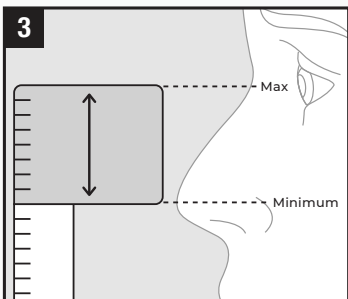


If this nose area falls within the shaded area of the measuring tool, the mask will most likely fit the length of your face.

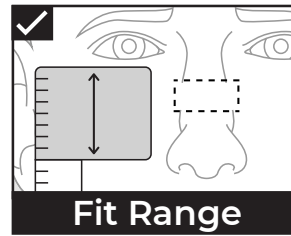

Fit Range

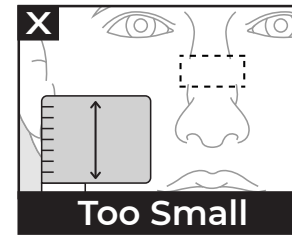

Too Small

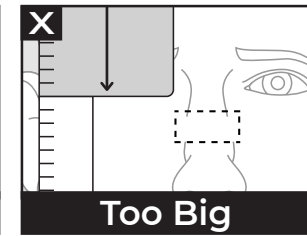

Too Big

