


# envo<sup>®</sup> pro Regular


## Half Mask Size Guide

### Sizing Instructions


Print out “L-Shaped Template” to the actual size provided (100% scale) and cut the measuring tool along the dashed lines.

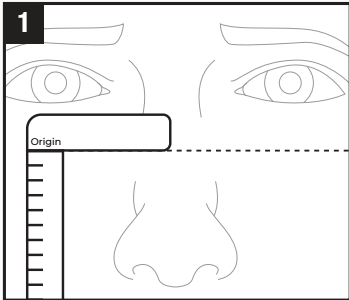

Using the provided L-shaped measuring tool, place the Origin line on the bridge of your nose where bone meets cartilage.

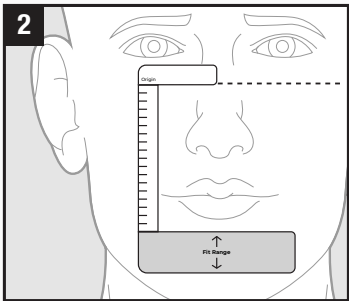


The tool should make an L-shape when applying it to your face.

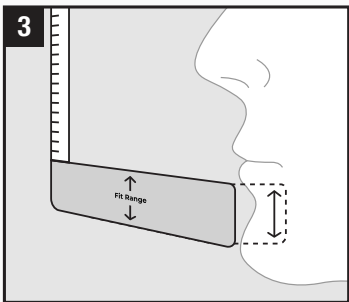


Check that the shaded area of the measuring tool covers your chin. If your chin is significantly above or below this region, the mask will most likely not fit your face.


M90157 Rev B

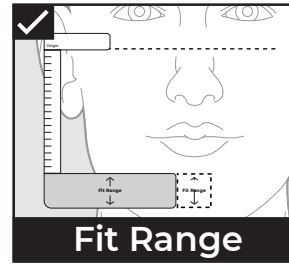


Fit Range

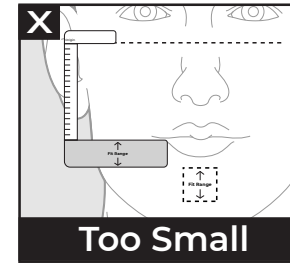

Too Small

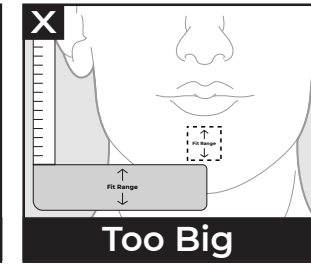

Too Big

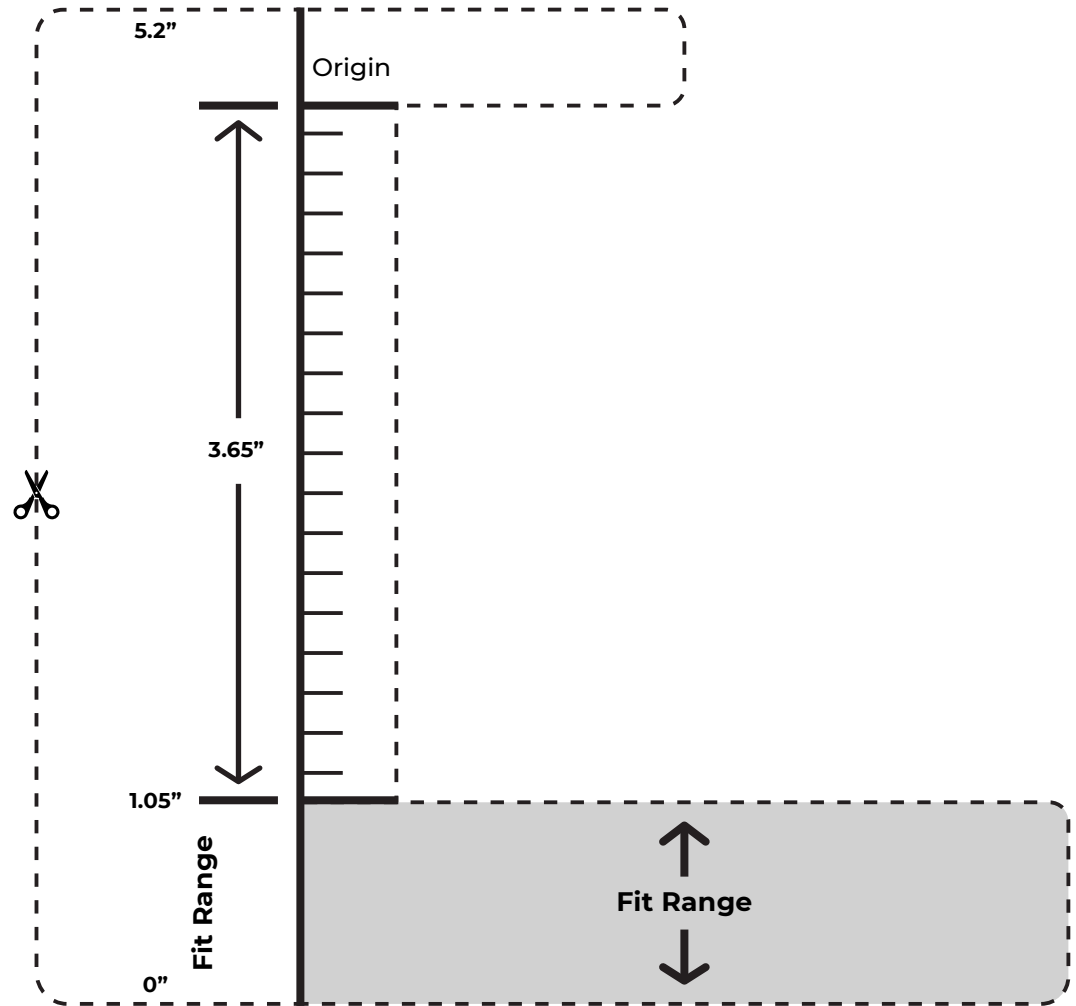

FOR REGULAR SIZE ONLY. PLEASE REFER TO THE XL SIZE GUIDE FOR THE NEXT SIZE UP.